

Posterior 2020 set USER MANUAL

1. This manual serves as a manual for the Posterior 2020 set teeth training model (hereinafter referred to as "Model").
2. This Model is not a toy and is only intended for use by persons over 18 years of age.
3. This Model is intended only for training of modelling of dental fillings using dental fillings modelling tools.
4. For training, use non-hardening plastic materials such as plasticine.
5. You can also use composite filling materials, but when they are used and photopolymerized the model becomes disposable and their removal without damaging the model is not guaranteed.
6. Do not interfere with the Model mechanically, do not use dissection tools. The Model is not specified for preparation methods training.
7. Store this Model at a temperature of 15-25°C, in a dark and dry place. Do not expose the Model to direct sunlight or temperatures higher than 45°C. When exposed to direct sunlight the Model may change its colour or its' structure can be damaged.
8. To clean the Model use clean water without using detergents. Do not leave it submerged in water for more than 15 minutes.
9. The model is made of acrylic resin, which may cause allergic reactions in some people. If you have had symptoms of an allergic reaction to acrylic resins in the past or if you want to reduce the risk of its occurrence, we recommend using it protective nitrile gloves when working with and handling the Model.

